INSTRUCTIONS FOR USE

EN

Name
Scarban® silicone sheets - Elastic and Light

Product Description

Scarban® silicone sheets are a proven technology for treating scars (hypertrophic and keloid) and preventing abnormal scar development after surgery or an accident. The medical silicone sheet is self-adhesive (with the exception of the silicone sheets glove and finger), washable and can be used for a long period of time.

Scarban® Silicone Cleaning Soap is a cleaning agent for silicone sheets.

Composition

- Textile layer: polyamide, elastane

- Medical grade silicones

Properties

- Scarban® silicone sheets ensure that scars become flatter, smoother and thinner and reduce complaints (like persistent redness, pulling of the scar, itching, restriction of points, movement, discomfort and cosmetic concerns).

The silicone sheet provides a semi-conductive (conducting) layer on top of the skin, which results in increased hydration of the skin (epidermis). This increased hydration ensures that the damaged scar skin recovers more quickly and if functional objections (complaints) are reduced.

Scarban® silicone sheets provide UV protection (UPF50) as standard.

Indications/Applications

Scarban® silicone sheets are suitable for:

- both old and new scars
- all skin types
- all ages
- treatment and prevention of abnormal scars (hypertrophic and keloid)
- treatment of burn scars
- treatment of surgical scars
- use under clothing and in compression therapy
- treatment of persistent functional complaints caused by the scar (persistent redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic concerns)

Instructions for use - Scarban® silicone sheets

(For instructions on how to use the glove and finger, see below)
For optimum results preferably start using Scarban® silicone therapy immediately after wound closure (after removal of sutures and the dressing).

It is recommended that Scarban® silicone therapy is built up gradually, up to the maximum of 23 hours of use per day. The purpose of building up gradually is to allow the skin to become accustomed to the silicone.

For the best possible result, it is preferable that the silicone sheets are used for a maximum of 12 hours and up to a maximum of 23 hours per day.

If necessary, Scarban® silicone sheet can be cut to the correct size (fig. 1).

Ensure that the silicone sheet extends two centimetres beyond the edges of the scar.

Ensure good adhesion and durability of the silicone sheet by:

- keeping the skin clean and dry before applying the silicone sheet

- removing the protective film from the back of the silicone sheet (fig. 2)

- drying: dry the scar, place the Scarban® silicone sheet over the scar and apply Scarban® silicone sheet (fig. 3)

- remove the protective film from the Scarban® silicone sheet when cleansing, perspiration or contact with water may cause the Scarban® silicone sheet to delaminate.

- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

How long should I use Scarban® silicone sheets?

- It takes at least 3-6 months for a scar to normalize.

- Use the Scarban® silicone sheet as long as the scar is raised, red or causes complaints.

- Consult your physician about the duration of the therapy.

Duration of use

- Each pack of Scarban® contains one or more silicone sheets.

- Used correctly and cared for properly (according to the instructions for use), each pack of Scarban® is sufficient for approximately 4 to 8 weeks of treatment.

- In the event of long-term therapy, several packs may therefore be necessary.

Special attention and precautions

- Keep out of reach of children.

- Scarban® silicone sheets are only suitable for external use.

- Use Scarban® silicone sheets only on healed, closed skin (after the scar has disappeared) and never on wounds.

- Dry the scar, place the Scarban® silicone sheet over the scar and apply Scarban® silicone sheet (fig. 3)

- If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage (fig. 5).

- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

Instructions for use - Scarban® silicone sheet: glove

(For instructions on how to use the glove and finger, see below)
Optimum results are achieved by starting to use Scarban® silicone therapy immediately after wound closure (removal of sutures and the dressing).

It is recommended that Scarban® silicone therapy is built up gradually, up to the maximum of 23 hours of use per day. The purpose of building up gradually is to allow the skin to become accustomed to the silicone.

For the best possible result, it is preferable that the silicone sheets are used for a maximum of 12 hours and up to a maximum of 23 hours per day.

If necessary, Scarban® silicone sheet can be cut to the correct size (fig. 1).

Ensure that the silicone sheet extends two centimetres beyond the edges of the glove.

Ensure good adhesion and durability of the silicone sheet by:

- keeping the skin clean and dry before applying the silicone sheet

- removing the protective film from the back of the glove (fig. 2)

- drying: dry the glove, place the Scarban® silicone sheet over the glove and apply Scarban® silicone sheet (fig. 3)

- remove the protective film from the Scarban® silicone sheet when cleansing, perspiration or contact with water may cause the Scarban® silicone sheet to delaminate.

- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

Instructions for use - Scarban® silicone sheet: Finger

The Scarban® silicone sheet is used for scar treatment on the fingers.

First determine the length of the silicone sheet that is needed. To do this, lay the finger on the sheet. There are several colored markings on the side of the sheet. If the sheet is too long, the silicone sheet can be cut to the correct size (fig. 1).

Secure the finger with a neutral, water-based adhesive or elastic bandage (fig. 5).

After removal of the silicone sheet, the finger may be treated with a hydrating cream.

Date of latest revision of instructions for use

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